LAMPE INFRAROUGE

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INFRARED LAMP

Thank you and congratulations for purchasing this examination lamp which has been the object of all our care and attention during its manufacture and quality control. This product will give you full satisfaction for many years. Before using it, you should read this notice carefully.

Legal Compliance

CE marking according to Directives 2014/35/UE, CEM 2014/30/EU and 93/42 CEE and the following harmonized standards: EN 60.598-1, EN 60.598-2-1, EN 60.598-2-25, EN 60.601-1, EN 60.601-1-2, EN 50.581-2-41 (ROHS2).

Uses

This lamp is specially designed for indoor lighting, it is recommended to use it with our various optional accessories as required. Place the infrared lamp at a distance of approximately 40 cm: the sensation of heat must always be agreeable. Sessions from 10 to 30 minutes are sufficient.

Recommendations

Before installing the lamp or carrying out maintenance (*for example: bulb replacement*), always unplug the lamp from the socket. Do not use the examination lamp in a humid atmosphere. When replacing the bulb please check the wattage: it must not exceed the authorized power rating. Do not close the ventilation openings on the lamp head. Bulb replacement: wait until the lamp has cooled sufficiently to avoid any risk of burns.

Maintenance instructions and cleaning

Mounting and connecting the lamp examination should be performed by qualified persons. Before any maintenance, always check that the lamp is unplugged. All cleaning products (*disinfectant, detergent, etc...*) can be used. Difficult stains or spots can be removed with an abrasive cream.

Installation instructions

- 1 Pull up the plastic cover from the base of the lamp
- 2 Screw the lamp onto the platform (*from the top*)
- 3 Replace the plastic cover

Exclusion of guarantee for flexible necks

 Do not bend the flexible neck too close to the rigid tubular end (*leave at least 40mm*) and do not bend it excessively.

 If the external power cable of this lamp is damaged, it must be replaced only by the manufacturer, their service agent or a similarly qualified person, in order to avoid any risk.

ZA Bertoire 36 avenue Jean Monnet
13410 Lambesc France • Tel : +33 442 928 976
Web : lid-lamps.com

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