LAMPE DE WOOD

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WOOD LAMP

Thank you and congratulations for purchasing this examination lamp which has been the object of all our care and attention during its manufacture and quality control. This product will give you full satisfaction for many years. Before using it, you should read this notice carefully.

Wood lamp, 9 Watts, black light, 370 nm wavelength. A portable wood lamp fitted with a ballast in the socket, to detect help in diagnosing fungal and bacterial infections of the skin, erythrasma or pigmentation problems (*vitiligo*).

Legal compliance

CE marking according to Directives 2014/35/UE, CEM 2014/30/EU and 93/42 CEE and the following harmonized standards: EN 60.601-1, EN 60.601-1-2, EN 60601-2-41, EN 50.581-2-41 (ROHS2).

Precautions for use

Do not look directly at the light source to avoid any risk for the eyes.

This lamp is specially designed for indoor lighting, it is recommended to use it with our various optional accessories as required.

Recommendations

Before installing the lamp or carrying out maintenance (for example: tube replacement), always unplug the lamp from the socket.

Do not use the examination lamp in a humid atmosphere. When replacing the tube please check the wattage: it must not exceed the authorized power rating.

Tube replacement: wait until the lamp has cooled sufficiently to avoid any risk of burns.

Maintenance instructions and cleaning

All cleaning products (*disinfectant, detergent, etc...*) can be used. Difficult stains or spots can be removed with an abrasive cream.

If the external power cable of this lamp is damaged, it must be replaced only by the manufacturer, their service agent or a similarly qualified person, in order to avoid any risk.

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