## C3700

CAMILLA HIDRÁULICA / HYDRAULIC TABLE / TABLE HYDRAULIQUE

### INSTRUCCIONES - INSTRUCTIONS

- \* Leer antes de utilizar por primera vez.
- \* Read carefully before the first use.
- \* A lire avant la première utilisation.

etiqueta CE

## CAMILLA HIDRÁULICA

### INSTRUCCIONES

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Nº de accionamientos del pedal: 24-35

## HYDRAULIC TABLE INSTRUCTIONS

Ecopostural thanks you for buying our products. The item you have just purchased has been manufactured according to high quality and security standard. Follow the below-mentioned instructions carefully in order to be able to make use of it for a long time. Click on to the **TECHNICAL INFORMATION** section in our website **[www.ecopostural.com](http://www.ecopostural.com)**

**Our company will decline any responsibility for a product damage occurred if ignoring our operating instructions. We will not be liable for any accident or harm resulting from an undue or negligent use.**

### 1) GENERAL INSTRUCTIONS

- Please read carefully the operating and safety instructions before using our products. These instructions must be adhered to. Keep the instructions booklet for further check-ups.
- Familiarize yourself with the precise use of the item.
- If the table is handled by unqualified people in an inappropriate way or not according to the intended purpose, the use of the examination table may be dangerous. Be careful when handling the table as there is a risk of being trapped by the mobile parts and because of sudden movements.
- Regularly check the table entirely to find any sign of malfunction or damage.
- Worn or damaged pieces must be solely replaced by original pieces by the manufacturer depending on the table model.
- All components parts or accessories should be fitted or fixed in according to the special manufacturer's instructions. Any spare / replacement component part different to the manufacturer's could be dangerous for the people involved in the table handling.

**\* Notice: The use of non original pieces or the alteration of the item will incur in a loss of validity of the manufacturer's guarantee, instructions and Declaration of Conformity.**

### 2) OPERATING INSTRUCTIONS

#### **a) SET UP**

The table must lean on a solid and level ground.

***Notice:*** in certain floor surfaces the table legs might leave marks after a prolonged contact. Check the technical specifications of the floor manufacturer to avoid any trouble.

This table is equipped with ø 125 mm **casters** suitable **for transporting the patients**.

There are 2 directional locking casters. For an easier driving of the table simply press down the *green lever* and the casters will be locked. Then lift the lever and the wheels will be released again.

The other 2 casters side are brake-equipped. To lock the casters press down the *white lever*, then lift the lever and the casters will release.

***Be careful*** not to lift the table up by holding it from the upholstery because this movement will cause a wind stroke and, consequently, the hydraulic system will be too winded and won't work properly.

#### **b) LOAD capacity**

This item has been tested to support a **MAXIMUM STATIC WEIGHT of 135 Kg** evenly distributed all over.

Absolutely avoid leaning on the very edge for support as it may tilt over.

If pressure on a reduced area exceeds the allowed weight, the table upholstery could split up. Keep the patients from getting on by their knees.

To **get on** the table the patient must seat at the middle of the table, raise his legs and lean on his side. Then, he can lie down. Weight must necessarily be distributed evenly and not concentrated on either side of the table.

When **getting down** the table different procedures apply depending on the position of the patient. Therefore, if he is lying down (prone position) he has to slide gently towards the middle of the table, lean on his side and go up by standing on his arms. He will then be able to put his legs out by the table's side and easily stand up.  
If the patient is lying upwards he will have to stand on his arms, get back to a seated position and then go down the table.

**\* *Caution:*** Never use the head or back rest sections to lean forward as these parts are not aimed at supporting such a pressure.

### c) HEIGHT adjustment

The central height of the table is operated by a **hydraulic pump**.

**SAFELY POSITION PATIENT BEFORE TABLE HEIGHT ADJUSTMENT.**

**Do not insert hands, feet or garments into the moving parts** of the elevating mechanism as this could result in serious injuries.

Press the **lever** with the foot in order to lift the table. With every stroke the table will go up a few centimeters. Raise the lever with the foot and the table will get down gently.

### d) SECTIONS adjustment

The **head-/backrest** section is operated by rack (2)

(2) Rack: Lift the section up to the required position and the rack will position itself automatically. To lower the section, first raise it up to a certain extend or completely (as appropriate) in order to release the rack.

*\* The hinges on both sides must be properly fixed and positioned at the same height.*

**NEVER adjust any of the table sections WHILE THE PATIENT IS LEANING ON.**

**Safely position your FINGERS AND HANDS AWAY from the upholstery edge** to avoid any accidental jamming.

### e) ACCESORIES Adjustment

The table you have just purchased is equipped with a set of **collapsible side chrome BARS**.

To **lift** the side bar grasp the upper handrail by any of the 2 grips and pull it up till you hear a locking click.

To **fold** the side bar pull the round knob (at the feet end) outwards for a proper unlocking, hold the side bar by the grips and take it down.

Always **hold the side bar by the grips** of the upper handrail.

Before folding the side bar, **MAKE SURE THAT NEITHER THE PATIENT OR THE THERAPIST GET TRAPPED** by their hands, feet, limb or any garment between the moving parts of the table.

## 3) MAINTENANCE

### a) Metal FRAME

For a proper operation of the table, you should **do the dusting** regularly with a cloth.

We also recommend to periodically **grease** the axles at the moving parts with a liquid grease spray. Lay the couch on its side and take off the white caps protecting the nuts at the axles of the framework. Slightly unscrew the nuts and spray the product around the loosen nuts. Leave to settle in this position for 12 hours. Tighten the nuts and put the protective white caps back on. Lay the table on the other side and repeat.

Periodically **check** the couch visually for any signs of damage and make sure that all bolts, nuts and other fasteners are tight.

*\* Please note that the repeated exposure of the table to high levels of humidity is not recommended.*

### **b) HYDRAULIC system**

Keep this piece free from any dust or grease.

**THE HANDLING OF THE PUMP IS TOTALLY FORBIDDEN. Any manipulation will invalidate the guarantee.**

If any malfunction or failure is suspected, contact immediately the reseller or Ecopostural.

### **c) UPHOLSTERY**

A wet cloth should be enough to do the dusting. Anyway, if the leather is not properly clean you can use a smooth sponge soaked with neutral soap water. Dry immediately afterwards. To prevent moisture from entering the seams, do not wash with abundant water.

It is highly recommended to use a towel cover like the A4469, A4459 or A4460 to protect the upholstery from sweat, massage creams or oils.

Use soapy water to clean the stains. **Avoid using alcohol, any solvent or stain remover** as the vinyl could be seriously damaged.

The use of infrared lamps might cause a cracking of the artificial leather. You should then put a cotton sheet or towel on the couch.

**\* For further details please CHECK THE ENCLOSED UPHOLSTERY INSTRUCTIONS.**

## **ECOPOSTURAL GUARANTEE**

All our equipment is covered by a **3 years** guarantee, from the original invoice date, against any manufacturing flaw or mechanical defect. It will be thus necessary to keep the invoice or any other purchasing document as proof of guarantee.

Wear and tear of the artificial leather will not be covered by the guarantee.

The following conditions will invalidate your guarantee: any damage or deterioration caused by misuse (e.g. inappropriate leather cleaning), by an undue unwrapping with a cutter (e.g. upholstery rips and tears) or by accident.

In those cases where the guarantee applies, Ecopostural will repair or replace the faulty part free of charge and return it at our expense within the EU territory.

For full details of the guarantee, please contact your supplier.

**PRODUCT** 

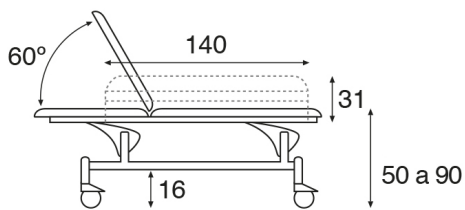
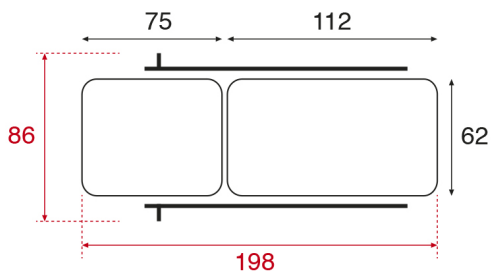
**MANUFACTURED BY ECOPOSTURAL, S.L.U.** ESB12433991  
Avda. Enrique Gimeno, 27 - 12006 Castellón de la Plana, **SPAIN**  
[www.ecopostural.com](http://www.ecopostural.com) / [info@ecopostural.com](mailto:info@ecopostural.com) / +34 964 20 01 53

The disposal of this product is subject to a specific EC Directive and it must be followed accordingly, as shown by the crossed skip symbol. Therefore, at the end of its useful life, you will have to apply to a specialized waste disposal centre.

Separate Collection of used products and packaging allows materials to be recycled and used again. Re-use of recycled materials helps prevent environmental pollution and reduces the demand for raw materials.

**TECHNICAL FEATURES C3700****General characteristics**

Hydraulic twin pillar white frame speciality table.

Standard upholstery dimensions: 62x188 cm (M44)

Recommended maximum weight: 135 Kg

Height variation: 50-90 cm

Hydraulic gas-spring assisted by foot pump.

Height adjustment with side misalignment < 20cm

Steel frame with white epoxy powder coating heated at 250°C

Gap of 16 cm to the frame.

Ø 125 mm castors: 2 directional castors and 2 castors with brake.

Side bars removable (A4700)

Drip pole stand included as standard.

Standard backrest type T15 - Square backrest. Without breather hole.

Head-/Backrest adjustable manually, by rack.

Approx. weight: 84 Kg

**Upholstery Features**

Fire-resistant Class M2 artificial leather, EC compliant.

Sanitized® anti-microbial and antibacterial protection.

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