



Jiangsu Suyun Medical Materials Co., Ltd
江苏苏云医疗器材有限公司

Add: No.1, Medicine Lane, Renmin Road, Lianyungang, Jiangsu Province, China
地址: 中国江苏省连云港市人民路制药巷1号 (222002)

Using Instruction

Product:

Sterile Otolaryngological Set for single use Code.488802

Components

1pcs Nasal Speculum, 2pcs Ear Funnel, 1pcs Tongue depressor, and 1pcs forceps, and 6pcs cotton tip wooden applicator, and 2pcs laryngological speculum and 1pcs Dental bibs

Product usage:

It is widely used in the ENT clinic examination, nasal speculum examine the nose, ear funnel could watch the inner ear, tongue depressor and laryngological speculum examine the mouth and throat.

Cotton tip applicator used for sampling and cleaning. Before operation, firstly check if package have broken, then examine whether all component are enough in the plate, then fix the dental bibs at the chest of people, to avoid pollution.

Attention

Do not use if the package broken. Do not reuse it.

Operation should be generally, so that the patient will not feel hurt.

Storage

Products should be stored in Cool, dry, clean environment

Expiration date

5 years after sterilization.

