



CryoPen|x+

Cryosurgery at its best



With the possibility to use both 8g and 16g cartridges CryoPen|x+ can treat all skin imperfections you will be confronted with in your clinic.

CryoPen®



Designed for >

DERMATOLOGISTS
GP'S / FAMILY DOCTORS
AESTHETIC CLINICS
PLASTIC SURGEONS
GYNECOLOGISTS / UROLOGISTS
PEDIATRICIANS
PODIATRISTS
VETERINARIANS

Blue dot
● Ø 1-3mm
Power:
■

White dot
○ Ø 2-6mm
Power:
■■

Blue dot
● Ø 4-8mm
Power:
■■■

Yellow dot
● Ø 7-20mm
Power:
■■■■

60mm Blue
● Ø 1-3mm
Power:
■

60mm White
○ Ø 2-6mm
Power:
■■

- full control with on/off switch
- extremely gas efficient
- reassuring refined design with aluminum housing
- comes with 6 applicators

For what kind of lesions ?

All benign skin lesions: Verrucae, skin tags, solar lentigo, pigmented spots, actinic keratoses, seborrheic keratoses, hemangioma, molluscum contagiosum, condyloma, plantar warts, and more...

What size of cartridges are used ?

With the 8g, the CryoPen|x+ has a maximum of 120 seconds of constant flow.

With the 16g, the CryoPen|x+ has a maximum of 240 seconds of constant flow.

A typical freeze on viral infections may last from 5 to 10 seconds for a small flat wart, and up to 45 seconds for a full thickness plantar wart. For general purposes, **most lesions take about 2 to 30 seconds**.

This is on the price list for CryoPen|x+

CryoPen|x+ : 6 applicators

With blue, white, green and yellow dot micro-applicator

+ additionally blue and white dot longer 60mm selection.

Delivered with forceps for cartridge opener/filter.

placement, user manual & carrying case.

8g/16g converter included.

This is what your colleagues say >
This is what was published in the press >

All our instruments



H&O | All rights reserved
© H&O Equipments 2018
EU Privacy Policy - US Privacy Policy

