

User manual

ADJUSTABLE SPIDER STRAP

REF 34018 Size: 203 x 51 x 5 Cm

REF 34082 Size: 183 x 46 x 5 Cm

Product introduction:

This product is designed to be used for spinal board with the use of spider strap for fixing patient. It can adjust the space between two straps vertically to better fit patients of different sizes. The product is reticular, strap ① for fixing shoulder and chest, strap ② for fixing the waist, strap ③ for fixing the thigh, strap ④ for fixing the crus. These straps are interrelated and adjustable vertically, and easy to operate.

Operation steps:



Figure 1



Figure 2

- 1) Place the patient in the spinal board which operated by professionals.
- 2) Fix the patient's body parts, and make adjustments for a better fit (adjust the space between two straps vertically). Check to ensure it is suitable after fixing. During the process of patient transfer, pay attention to the tightness of straps, and make appropriate adjustments if necessary.

Notice:

The operators must be first-aid professionals, or trained and qualified personnel.

Read the user manual carefully before use.




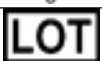


Check to see if the product is intact before use.

Know about the patient's condition before use, and pay attention to the injured part during fixing.

Warning:

This product must be operated by first-aid professionals, or it may cause other harm to patients.



Symbols	Symbols description
	Please read instructions carefully
	Keep away from direct sunlight
	Store in a dry place and avoid humidity
	Lot number
	Manufacturer
	Product code