# Spengler

FR

## OTOSCOPE SMARTLIGHT

FR

### MODE D'EMPLOI

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**HONSUN (NANTONG) Co., Ltd.**  
Address: No.8, Tongxing Road, Economic&Technical  
Development Area, Nantong City, Jiangsu, P.R.China

**EC REP SHANGHAI INTERNATIONAL HOLDING CORP. GMBH (EUROPE)**  
Address: Eifflstrasse 80, 20537 Hamburg GERMANY

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## EN OTOSCOPE SMARTLIGHT EN

### INSTRUCTION MANUAL

An instruction of otoscope for ENT specialists. It is an internally powered device. Please read through this brochure before use, and keep it handy for future reference.

#### Symbols:

Manufacturer

Authorized Representative in the European community

Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. the device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. please follow Local Ordinances or Regulations for disposal.

Type B Applied Part

Attention, consult accompanying documents

### SAFETY INFORMATION

To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

#### WARNINGS & CAUTIONS:

- Do not use the device where it may be exposed to flammable gas.
- Protect this device from water, moisture, impact, dropping, direct sunlight, heat and fire.
- Do not touch the lens directly by hand or something hard.
- Please use the bulb type specified in this instruction brochure.

- Keep the device out of the reach of unsupervised infants and children.
- Do not disassemble or attempt to repair the device or components.

### GENERAL

#### Description:

The otoscope is a classic medical device in ENT. As a traditional diagnostic otoscope, SMARTLIGHT adopts Direct Illumination Technology. It is used to view the tympanic membrane and external ear canal to diagnose outer and middle ear pathologies.

#### Technical Specifications:

- Enhanced XHL Xenon Halogen Technology
- 3X magnification for the viewing window.
- #108 bulb used.

### HOW TO USE THIS DEVICE

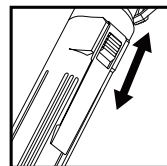
fig.1

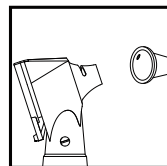
fig.2

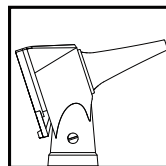
fig.3

- Turn on (or off) the device by pushing downwards (or upwards) as **fig.1**;
- Select a reasonable tip for patient's ear canal and push it onto the hand and screw the tip clockwise, refer to **fig.2**. Generally, the tip size of 4mm is for adult and 2.4mm is for children;
- Put the otoscope with a tip into the ear canal of patient (**fig.3**), and examine through the viewing window which is with 3X magnification;

#### ATTENTION:

**The otoscope should only be used to examine ear when a tip is fitted. It is an intermittent operated device, 1 minute ON, 5 minutes OFF.**

### MAINTENANCE

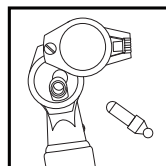
fig.4

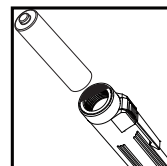
fig.5

#### Changing Spare Parts:

##### 1. Changing the bulb (fig.4)

- Turn off the power and screw the viewing window off the slot on bottom of the head;
- Pull out the old bulb in the metal support and replace with new one;
- Screw the viewing window back on to the slot.

#### ATTENTION:

- **Please use the specialized bulb (Model :# 108) for this device.**
- **After the bulb burns out, the temperature will remain high for approximately 5 minutes. Please wait for a moment to avoid injury.**

##### 2. Changing the batteries (fig.5)

- Screw the head off the handle;
- Remove the old AA batteries, and replace with new ones. Ensure the anode of the batteries points to the head of the device;
- Screw the head back on the handle.

#### ATTENTION:

- AA alkaline batteries are recommended.
- Replace all batteries simultaneously and do not use alkali manganese batteries with lithium as bulb life will be reduced.
- Do not leave worn batteries in the handle.
- Please take out batteries from the device if not in use for a prolonged period.
-  Batteries should be disposed of in accordance with local regulations.

#### Cleaning and Disinfection:

- Clean the surface of device with 75% alcohol solution at least once a week, avoiding the magnifying glass (viewing window) in the head. And corrosive or polishing detergent is prohibited.
- Disinfect the tips of otoscope with saturated steam for 20 minutes at 134°C.

#### Attention:

**Do not soak, autoclave or let water dip into the device!**

### PERIODIC SAFETY CHECKS

Preventive inspection and maintenance to be performed including the frequency of such maintenance. The following safety check should be performed at least every 12 months by a qualified person who has adequate training, knowledge, and practical experience to perform these tests.

- Inspect the equipment and accessories for mechanical and functional damage.
- Inspect the safety relevant labels for legibility.
- Verify that the device functions properly as described in the instructions for use.

### SPECIFICATIONS

|                                |                                                                                                                              |
|--------------------------------|------------------------------------------------------------------------------------------------------------------------------|
| <b>Illumination</b>            | Direct Illumination                                                                                                          |
| <b>Luminance</b>               | 24 Lumen                                                                                                                     |
| <b>Magnification</b>           | 3 x                                                                                                                          |
| <b>Tip Sizes</b>               | 2.4mm, 3mm, 4mm, 5mm                                                                                                         |
| <b>Bulb</b>                    | Specified Model: #108<br>2.5v, 0.75A, 3050K, 25hrs                                                                           |
| <b>Batteries</b>               | 2 AA alkaline batteries                                                                                                      |
| <b>Operation</b>               | <b>Temperature</b> 10°C ~ 35 °C<br><b>Humidity</b> 30% ~ 75%<br><b>Air pressure</b> 700hPa ~ 1060hPa                         |
| <b>Storage &amp; Transport</b> | <b>Temperature</b> -20°C ~ 70 °C<br><b>Humidity</b> 10% ~ 90% (without condensation)<br><b>Air pressure</b> 500hPa ~ 1060hPa |
| <b>Guarantee</b>               | <b>2-years</b> (from date of purchase)                                                                                       |
| <b>Standards</b>               | MDD 93/42/EEC Class I<br>IEC 60601:2012, IEC 60601-1-2:2014                                                                  |

### ELECTRICAL CLASSIFICATION

Equipment Classification with respect to protection from electric shock: Internally powered equipment. Degree of protection from electric shock: TYPE B. Equipment not suitable for use in the presence of flammable mixture: Yes. Mode of operation :intermittent operation.

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**HONSUN (NANTONG) Co., Ltd.**  
Address: No.8, Tongxing Road, Economic&Technical  
Development Area, Nantong City, Jiangsu, P.R.China

**EC REP SHANGHAI INTERNATIONAL HOLDING CORP. GMBH (EUROPE)**  
Address: Eiffeistrasse 80, 20537 Hamburg GERMANY

*The manufacturer reserves the right to make technical changes without notice in the interest of progress. Prior notices will not be given in case of any amendments within this manual.*

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Distribué par SPENGLER SAS - ZI La Limoise